

Relax with a

relaxa

mobility 

Bathlift



- Easy to fit
- Battery powered NOT mains electricity
- Audible alarm when battery is low
- Remote control as standard
- Thermometer for water temperature
- Does not obstruct the bath
- Provides independence for those that find the bath difficult to enter or leave.

How does it work

1 Simply push in the re-chargeable battery.



Feed out the belt and slip on belt sleeve. **2**



3 Locate the floor anchor rod into the floor bracket and tension.



4

Sit on the belt and by touching a button, be gently lowered into the bath.



Relax and enjoy the luxury of a good soak. 5

6

Touch a button and be gently raised out of the bath.



Audible alarm and graphics when battery is low. In addition there is a remote hand control and a temperature probe to register the water temperature and avoid scalding.

Battery Care Instructions

Charge the battery for a minimum of **15 hours** before first use. We recommend a **12 hour** charge every week as frequent charging will maximise overall battery life.

Only use the approved Relaxa charger to recharge the battery pack.


Do not allow the battery pack to become wet.

Do not dismantle the battery pack or attempt to poke foreign objects into either connector.

When not in use the battery should ideally be stored between 20-25°C.

The battery pack should be removed from the Relaxa unit when not in use for more than a few days.

Batteries not in use will self-discharge over long periods, so they will need recharging.

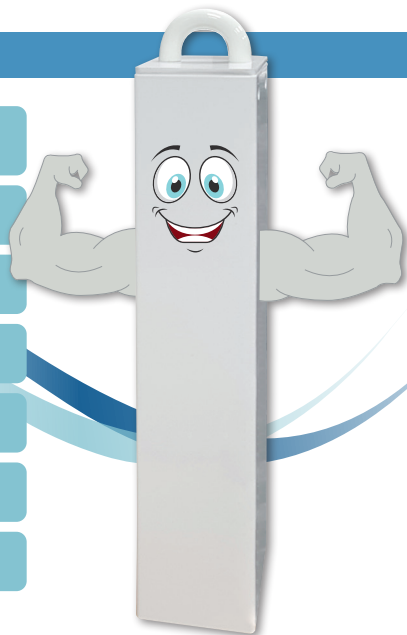

Technical Specifications

Thermometer	The unit has a thermometer to test the temperature of the water.
Power	Two 12 volt rechargeable lead acid batteries provide a 24 volt DC supply to the motor. Indicator lights in green and red shows the battery is in operation and the battery condition. A red light will appear to indicate that the battery is low together with an audible alarm. An empty battery symbol will appear when the battery is too low. The bath-lift will not work in the down position but will bring you back out of the bath.
Battery Charger	A managed 24 volt 300MA charger is provided.
Maximum Load	127 kg (20 stone)
Controls	“UP” and “DOWN” buttons are located on the front of the bath-lift. A hand control with “UP” and “DOWN” buttons is also provided.
Belt Reversal	The Relaxa is designed with an anti-belt reversal mechanism.
Belt	Reinforced Polypropylene
Dimensions	The unit is slim and fits neatly at the side of the bath. The length is 55.5cm, the height is 32.5cm and the depth is 11.8cm
Weight	The unit itself weighs 11kgs. The box when packed to include other items weighs 15.9kgs
Manufactured	The complete unit is manufactured in the United Kingdom

relaxa
mobility

For further information please contact

1 Kites Park, Summerleys Road,
Princes Risborough, Buckinghamshire, HP27 9PX
t: 01844 397044

www.relaxamobility.co.uk

Registered in England and Wales: 12076361